

# Diaclean

REF	Content
H19170	- 250 mL sodium hypochlorite solution

## GENERAL INFORMATION

<b>Method</b>	Manual cleaning
<b>Shelf life</b>	3 years from date of production
<b>Storage</b>	+15-25 °C

The Diaclean can for example be used to manually clean the aperture of a heamotology analyzer.

## REAGENT COMPOSITION

Sodium hypochlorite (<5%)  
 Distilled water

## REAGENT PREPARATION

Diaclean is ready to use.

## STORAGE AND STABILITY

Diaclean:

- must never be subjected to temperatures below 15°C and above 25°C.
- can be stored for 120 days (16 weeks) after opening.
- can be stored for 3 years (see expiration date on the product label) since the date of manufacture.

## WARNINGS AND PRECAUTIONS

- Please read the Safety Data Sheet (SDS) before use.
- Do not use directly after transport or directly after handling. Wait a few minutes.
- Cleaning solution can cause irritation of the eyes, skin and mucous membranes.
- In case of contact, rinse with plenty of water and seek immediate medical attention.
- Do not use the cleaning solution beyond the expiry date or beyond the open bottle time.
- Do not transfer / mix cleaning solutions whether they are of the same type or batch.
- Do not reuse an empty container as this may distort the results or damage the machine.
- Do not use the product in case of deterioration of the protective packaging.
- If the cleaning solution has been accidentally spilled, wipe it off immediately with a damp cloth.
- Do not use the product if there is any sign of deterioration (turbidity, color change, etc.)

## WASTE MANAGEMENT

Waste is disposed of according to local regulations.

## USED SYMBOLS



Dispose of the tests and packaging appropriately

